Instructions for Use [EN]

D-Dimer PBS Diluent REF AR037K

For *In vitro* Diagnostic Use.

1. Intended use

For dilution of samples and calibrators to enable analysis with in vitro diagnostic reagents for use in haemostasis diagnostics. Intended to be used by professional laboratory personnel.

2. Components

D-Dimer PBS Diluent consists of:
10 × 5 mL phosphate buffered saline solution and preservatives.

3. Warnings and precautions

Wear suitable clothing for protection. Avoid contact with skin and eyes. Do not empty into drains. Waste must be disposed of in accordance with local regulations.

D-Dimer PBS Diluent contains sodium azide (less than 0.1%) and 2-methylisothiazol-3(2H)-one (less than 0.0015%) to prevent microbial growth; use proper disposal procedures.

EUH208: Contains 2-methylisothiazol-3(2H)-one. May produce an allergic reaction.

EUH210: Safety data sheet available on request.

4. Preparation

Ready to use.

5. Storage and stability

Store at 2 - 8 °C. After opening, stable for 4 weeks at 2 - 25 °C in the closed original vial, provided no contamination occurs.

6. Material required but not provided

- Analyser with turbidimetric detection (refer to the Instructions for Use associated with the D-Dimer Latex LRT assay for valid instruments) and pipettes.
- Reagents.
- Calibrators and samples intended to be diluted.

7. Procedure

This device is an accessory to in vitro diagnostic reagents. For procedures, refer to the Instructions for Use of the D-Dimer Latex LRT reagent and to the instrument-specific application sheet. Note that if D-Dimer PBS Diluent is used in combination with other devices not provided by the manufacturer, the combination must be validated in accordance with Regulation (EU) 2017/746.

8. Reporting of incidents

Any serious incidents that occur in relation to this device shall be reported to the manufacturer as well as the national competent authority in which the user is established.

9. Additional information

The e-IFU (other languages) and the safety data sheet are available at www.hyphen-biomed.com. A paper copy of these Instructions for Use is available on request. Contact your local distributor.

For customer support and instrument-specific application sheet, please contact your local distributor.

10. Definition of symbols

Manufacturer

Unique device identifier

CE mark

Consult electronic instructions for use

In vitro diagnostic medical device

Use-by date

Catalogue number

Temperature limit

Batch code

11. Revision history

Version	Changes to previous version
4.0	Added translation to Danish, Finnish, Norwegian and Swedish in electronic instructions for use.

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