

Fib Clauss

Stable 7 days
after reconstitution

24 months stability

Traceable to
international standard

Fib Clauss is a functional clotting assay used for quantification of fibrinogen in citrated plasma. Fib Clauss reagent utilises the Clauss technique, where a high concentration of thrombin is added to pre-diluted plasma and the clotting time is measured.

The clotting time depends on the level of functional fibrinogen in the sample. Fib Clauss reagent has a large linear range, up to 8 g/L, limiting the number of re-analyses.



- ▶ Linearity up to 8 g/L
- ▶ High stability
- ▶ 100 UNIH/mL Thrombin
- ▶ Calibrated against SSC/ISTH Secondary Coagulation Standard

sales@nordicbiomarker.com
www.nordicbiomarker.com



For downloads, please visit nordicbiomarker.com

DETAILS

Product form	Lyophilised
Wavelength	For instrument specific settings, consult instrument applications.

ANALYTICAL PERFORMANCE CHARACTERISTICS

ACL TOP 700 CTS has been used for these measurements.

Measuring range	0.6–8.0 g/L		
Precision	Sample	Repeatability	
		Mean fibrinogen g/L	CV%
	Level 1	1.23	3.2
	Level 2	2.45	4.0
No interference with	<ul style="list-style-type: none"> ▶ Bilirubin <5 mg/dL ▶ Dabigatran <0.0475 mg/dL ▶ Triglycerides <375 mg/dL ▶ Unfractionated heparin <330 U/dL 		

STABILITY AND STORAGE

Storage	2–8 °C
Shelf-life	24 months at 2–8 °C
After reconstitution	7 days at 2–25 °C

ORDERING INFORMATION

PRODUCT NUMBER	PRODUCT NAME	CONTENT
C3103	Fib Clauss 5 mL	5 mL Reagent

ASSOCIATED PRODUCTS

PRODUCT NUMBER	PRODUCT NAME	CONTENT
C3123	Fib Clauss Calibrator	1 mL
C3109	Routine Normal Control	1 mL
C3110	Routine Abnormal Control	1 mL
C3112	Specialty Normal Control	1 mL
C3113	Specialty Abnormal Control	1 mL
C3135	Sodium Chloride Diluent 5 mL	5 mL
C3137	Laboratory Water 5 mL	5 mL